



Cleaning guide

to keep Polydentia's
Clipon & L-Clipon
suitable for a safe and optimal use



Clipon &
L-Clipon

Cleaning and Disinfection Instructions

Preliminary operations:

- For safe and optimal use, always clean and disinfect your product prior to use.
- Check and inspect your equipment regularly.
- Materials may alter over time. Exposure to chemicals can accelerate this deterioration. Inspect products for any damage prior to each use.

Cleaning and disinfection:

Products should be cleaned as soon as possible after use to prevent organic matter from hardening

1. Disassemble the shield from the Clipon
2. Remove visible debris or organic matter with a soft disposable wipe. This must be done by scrubbing with warm tap water and soap or detergent.
3. Repeat cleaning step 2 if there is still debris.
4. Rinse all parts with warm tap water.
5. Wipe shield using an appropriate disinfecting solution, following the manufacturer's instructions. Commonly used disinfecting agents based on Ethanol, Hydrogen peroxide, Phenol, Sodium hypochlorite, Calcium hypochlorite can be used for a superficial disinfection of the Clipon frame and shields.
6. Dry all the components using a soft microfiber cloth.

IMPORTANT:

Clipon and shield have always to be disassembled before reprocessing (cleaning, packaging and storage).

DISCLAIMER:

This high-quality product is manufactured by Polydentia SA, in Mezzovico-Vira, Switzerland. The content of this packaging has been inspected, tested and approved by our quality officers.

Materials used during our manufacturing process are designed to provide durability and performance. Polydentia constantly looks for the highest quality plastics for its products; nevertheless, please consider that plastics may alter over time. Exposure to chemicals can accelerate this deterioration.

Due to various chemical compositions of available disinfecting agents on the market, we suggest testing the disinfecting agent on a small surface of the device before proceeding with the complete disinfection.

For the longest lifetime of your premium quality product always clean it following these instructions.



 Polydentia SA
via Cantonale 47
6805 Mezzovico-Vira
Switzerland

phone: +41 91 946 29 48
fax: +41 91 946 32 03

info@polydentia.ch
www.polydentia.ch

