Sterilization of single use products

polydentia
swiss manufacture 

en - ENGLISH	2	da - DANSK	12
de - DEUTSCH	3	cs - ČEŠTINA	13
fr - FRANÇAIS	4	tr - TÜRKÇE	14
it - ITALIANO	5	el - ΕΛΛΗΝΙΚΑ	15
es - ESPAÑOL	6	bg - БЪЛГАРСКИ	16
pt - PORTUGUÊS	7	hu - MAGYAR	17
no - NORSK	8	lv - LATVIEŠU VALODA	18
et - EESTI KEEL	9	lt - LIETUVIŲ KALBA	19
fi - SUOMI	10	ro - LIMBA ROMÂNĂ	20
sv - SVENSKA	11	sk - SLOVENČINA	21

Sterilization of single use products

Basic Principles

Endo-Stops and ID-Rings are single-use only and cannot be reprocessed. Endo-Stops and ID-Rings are supplied non-sterile and must be sterilized before use. Ensure equipment used is serviced and checked regularly, and manufacturer's instructions are followed.

The user is responsible for the sterility of Endo-Stops and ID-Rings and to ensure the following as well:

- Only procedures that are sufficiently validated specifically for devices are used for sterilization.
- The equipment used is regularly maintained, checked and calibrated.
- The instructions regarding the equipment must be respected at all time.
- The user has to be trained adequately.
- In addition to these instructions, please observe the legal regulations valid in your country as well as the hygiene regulations of the dental practice.

Packaging

Packaging can influence the attainment of sterilization conditions. We recommend the use of sterilization pouches/ reels compliant with ISO 11607-1 and suitable for steam sterilization. The packaging shall be large enough to avoid stressing the sealing steam.

Warning! After the heat sealing process, the sealing seam shall be checked visually for any defects. In case of defects, the packaging must be opened and the product repacked and sealed

Sterilization

Sterilize Endo-Stops and ID-Rings only in a steam autoclave using distilled water and at the temperature specified on the relevant symbol. Do not use chemical, cold sterilization or dry heat sterilization.

Steam sterilization shall be validated according to ISO 17665 series. The autoclave shall be compliant with EN 13060.

- Follow carefully the instructions for use of the autoclave.
- Inspect the autoclave regularly according to the instructions for use. Service and clean the device frequently, according to the instructions for use.
- Ensure that maximum load is not exceeded.
- Place all components in a sterilization disposable pouch (refer to "Packaging").
- Endo-Stops and ID-Rings shall not touch the autoclave walls, since the temperature there might be higher.
- We recommended to use always the following cycle (Prion Cycle):

Temperature of sterilisation: 134°C

Time of sterilisation: 20 min

Drying time: 20 min

- Remove Endo-Stops and ID-Rings from the autoclave immediately after sterilisation.
- Check the integrity of packaging and products.

Storage

- Store sterilized product in dry and clean conditions at ambient temperature.
- Keep product sealed in the sterilization pouch until ready to use.

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