

unica anterior



powered by:


STYLE ITALIANO™

the one
solution
for anterior
restorations

matrix
systems



polydentia
swiss manufacture 

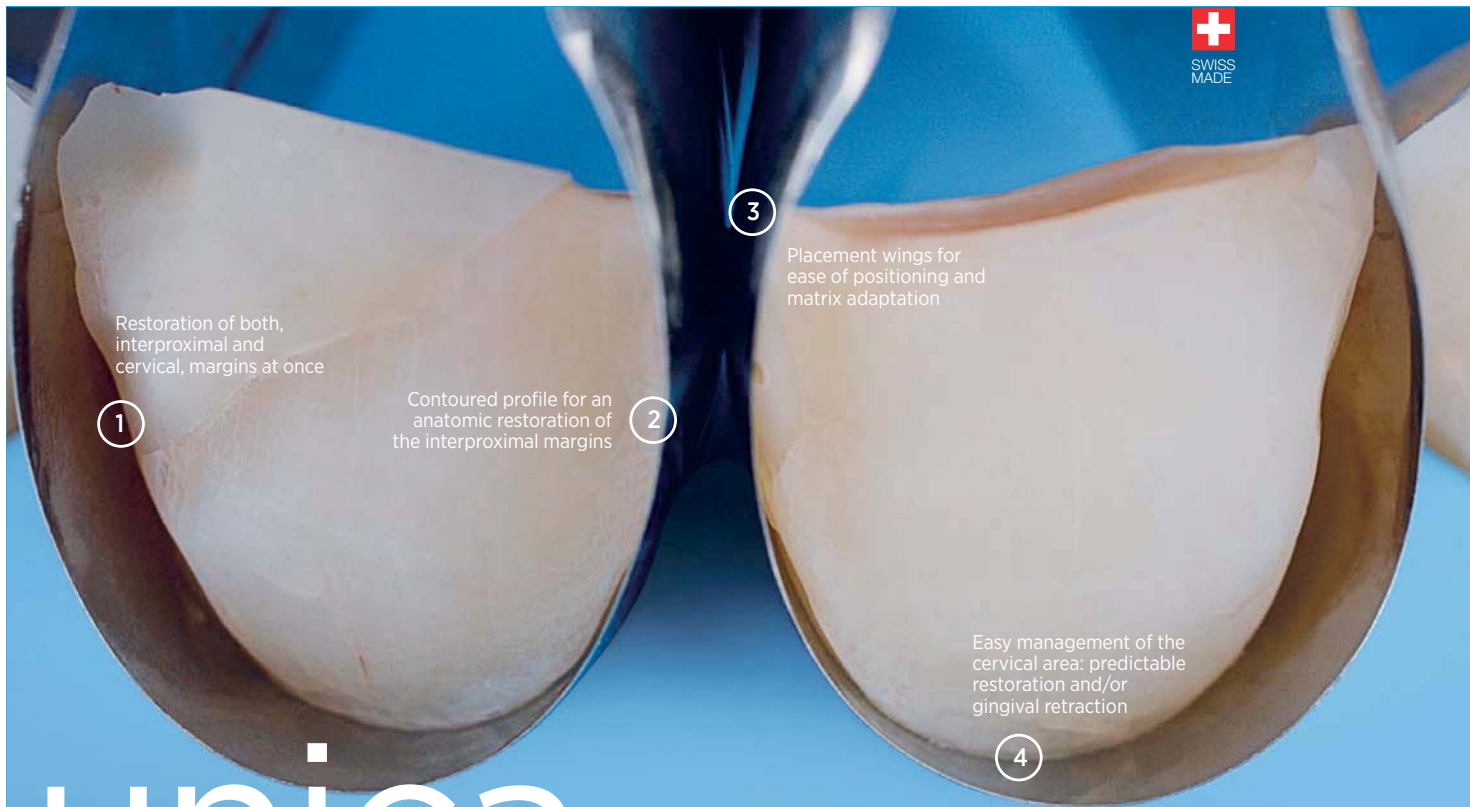

1 Restoration of both, interproximal and cervical, margins at once

2 Contoured profile for an anatomic restoration of the interproximal margins

3 Placement wings for ease of positioning and matrix adaptation

4 Easy management of the cervical area: predictable restoration and/or gingival retraction

unica anterior



"In today's conservative dentistry, the choice of products and the techniques we apply everyday must be predictable and cost-effective, incorporating clear-cut advantages to our daily procedures. Unica is the solution to everyday anterior restorations, whether it be class III, IV, V, shape modifications, and/or composite veneers in direct stratification technique. One effective matrix for everyday anterior aesthetic and functional results."

Dr. Jordi Manauta, DDS
Head of Styleitaliano Idea Factory
Sestri Levante, Italy

Unica is the simple and ideal matrix for anterior restorations such as class III, IV, V, direct stratification composite veneers, and shape modifications.


Thanks to its countoured shape, Unica adapts correctly to the different morphologies of anterior teeth and makes it possible to restore interproximal and cervical margins at once, even in presence of rubber-dam or gingival retraction cords, thus reducing chair time significantly. The placement wings allow fast and efficient matrix positioning.

Furthermore, Unica anterior matrix once positioned allows to easily visualize the final shape of the restoration, even before starting the procedure.



Composite veneers

for more information

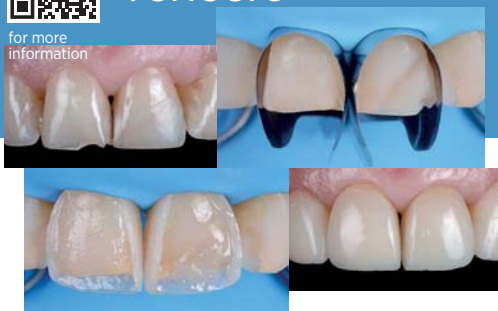

kit



Unica Introkit anterior - REF 6900

- 35 pcs Unica anterior matrices
- 40 pcs myWedge XS
- 40 pcs myWedge S
- 40 pcs myWedge M
- 1 pc myCustom Resin, 3 ml syringe
- 7 pcs needle tips gauge 18 form myCustom Resin
- 1 pc myQuickmatrix Forceps

refill



Unica anterior matrices
REF 6912 12 pcs
REF 6950 50 pcs



REF 5703
myQuickmatrix Forceps



REF 7101
myWedge
Extrasmall (XS) 100 pcs



REF 7102
myWedge
Small (S) 100 pcs



REF 7103
myWedge
Medium (M) 100 pcs



REF 6811
myCustom Resin, 3ml Syringe & 10 pcs. needle tips gauge 18

Class III restoration



In case of a Class III cavity where the proximal margin is missing, Unica is very useful as it allows the clinician to visualise the final shape even before proceeding with the restoration.

Dr. Giuseppe Chiodera
Brescia, Italy



for more information

Class IV restoration



For this Class IV restoration, Unica helps to restore the cervical and interproximal profiles anatomically and in a very fast way.

Dr. Dan Lazar
Oradea, Romania



for more information



Unica anterior matrices

myCustom resin



myQuickmatrix Forceps



myWedge

polydentia
swiss manufacture

Composite veneers



Thanks to the new matrix Unica we have the possibility to restore at the same time the cervical and interproximal margins with significant advantages in terms of shape management and predictability of the restoration.

Dr. Monaldo Saracinelli &
Dr. Antonino Nicolò
Italy



for more information

Shape modification




The unique design of Unica makes this matrix an essential and very efficient tool since it enables us to work both in the interproximal and cervical areas at the same time.

Dr. Carlos Fernández Villares - Madrid, Spain



for more information

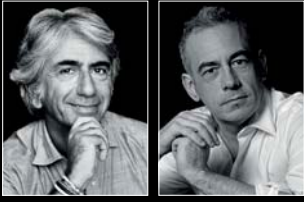

Styleitaliano

founded by
Dr. Walter Devoto and
Prof. Angelo Putignano

Style Italiano is a group of passionate clinicians and researchers, who are in constant development of materials and techniques for aesthetic and conservative dentistry. Their philosophy is based on developing solutions to make excellence in aesthetic and functional dentistry attainable by all.

For Polydentia, the collaboration with Style Italiano is more than just a source of inspiration. It is an opportunity to constantly renew its commitment to put excellence in Swiss manufacturing at the service of dentistry worldwide.



Visit “Unica
Community”
to share
Unica anterior
clinical cases

Follow us
#thinkunica



Polydentia SA
via Cantonale 47
6805 Mezzovico-Vira
Switzerland

phone: +41 91 946 29 48
fax: +41 91 946 32 03

info@polydentia.ch
www.polydentia.ch

