Quick Guide

Improved version:

three layers sewn together • time-saving • flexible • resistant!

Three-layer fiber-glass band, 2 mm wide, especially developed for post-orthodontic retainers.

An alternative to metal retainers, allows the fast and simple realisation of highly aesthetic orthodontic reinforcement, comfortable and less invasive for the patient.

Only for professional use

Made in Switzerland

POLYDENTIA SA
Swiss Products for Fine Dentistry

6805 Mezzovico-Vira
Switzerland

Phone +41 (0)91 946 29 48

Fax +41 (0)91 946 32 03

Web www.polydentia.com

E-mail info@polydentia.com

Since 1989 Polydentia is known for its complete product range consisting of glass fibre bands for the splinting of post-traumatic or periodontally loosened teeth, for fabrication of long-term temporary bridges and glass fibre posts.

Fiber-Splint Ortho Evolution has been especially developed for post-orthodontic retainers and constitutes the perfect combination between aesthetics and resistance.

The band is 2 mm (.08") wide and consists of three fibre-glass layers sewn together one over the other. The individual layers no longer have to be applied onto each other saving valuable chair time. The multi-layer version maintains its flexibility, thus it can be easily adapted and fixed to the palatal/lingual surfaces and in the interdental area, guaranteeing stabilization of the teeth. Retention in the interdental area is made easier also by using the special application clips, provided with Fiber-Splint Ortho Evolution.


INSTRUCTIONS FOR USE

1. Place a rubber dam.


2. Clean all involved teeth with non-fluoride polishing paste.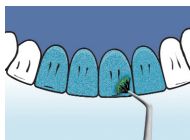


3. Etch the palatal/lingual and approximal surfaces of the teeth for 60 seconds then rinse and blow dry.

4. Coat the etched surfaces with an enamel-dentin bonding agent.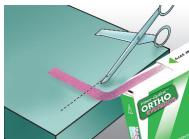


5. Cut a strip of Fiber-Splint Ortho Evolution according to the desired length.

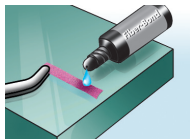6. On a glass plate, soak the strip in a drop of bonding agent and let the fibre glass strip absorb the liquid completely; wipe off any exceeding bonding material.

7. Apply the soaked strip on the etched and bonded surfaces and hold it in place by fixing the application clips "Clip&Splint" in the interdental area. Make sure that no air bubbles remain between the glass fibre strip and the tooth.

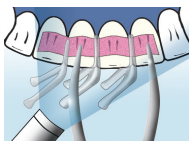8. The strip is kept in position while polymerized 30-40 seconds per tooth.

9. Any rough areas can be smoothed by applying a thin layer of flowable composite. Light-cure, then finish and polish the retainer. Remove the rubber dam and check the occlusion. Two or more layers of Fiber-Splint Ortho Evolution can be applied using the same technique.

Articles to reorder:

REF 5979 Fiber-Splint™ Ortho Evolution (width 2 mm / .08", length 60 cm), 5 application clips

REF 5975CS 30 application clips "Clip&Splint"

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