



Instructions for use

# hy-drop forte

hygienic liquid dispenser  
designed for special  
chemical agents

instruments  
and  
accessories

 polydentia  
swiss manufacture 

also available:

REF 5411/V  1 Hy-Drop violet + 5 membranes

REF 5411/B  1 Hy-Drop blue + 5 membranes

REF 5411/G  1 Hy-Drop green + 5 membranes

REF 5411/Y  1 Hy-Drop yellow + 5 membranes

REF 5411/O  1 Hy-Drop orange + 5 membranes

REF 5415 5 membranes, refill pack

Compatible with aggressive chemical substances:

REF 5412  1 Hy-Drop Forte black + 5 membranes



For professional use only  
REV. 05.09.2021



**Polydentia SA**

Via Cantonale 47 • 6805 Mezzovico-Vira • Switzerland

Phone +41 91 946 29 48 • Fax +41 91 946 32 03

[polydentia.ch](http://polydentia.ch) • [info@polydentia.ch](mailto:info@polydentia.ch)

Made in Switzerland




Pour the liquid into the container (max 4ml).



Position the membrane.



Press gently on the membrane.  
Hy-Drop releases the liquid drop by drop.

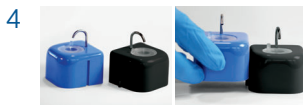

Can be interconnected.



Scan QR for instructions for use &  
Cleaning and Sterilization Guide  
or go to [www.polydentia.ch](http://www.polydentia.ch)

Compatible with - Kompatibel mit - Compatible avec - Compatibile con - Compatible con:

Description	Registered trademark of
ChKM-Lösung	Adolf Haupt & Co. GmbH
Fokaldry	lege artis GmbH
Orthoskavident® C	Kettenbach GmbH & Co. KG
Tubulicid Red Label	Dental therapeutics AB
Tubulicid Blue Label	Dental therapeutics AB
Tubulicid Plus Endo	Dental therapeutics AB
Cavidex	Detax GmbH & Co. KG

#### en - English

Polydentia does not guarantee compatibility with substances other than those listed in the table.

#### de - Deutsch

Polydentia garantiert keine Verträglichkeit mit anderen Stoffen als den in der Tabelle aufgeführten.

#### fr - Français

Polydentia ne garantit pas la compatibilité avec des substances autres que celles énumérées dans le tableau.

#### it - Italiano

Polydentia non garantisce la compatibilità con sostanze diverse da quelle elencate nella tabella.

#### es - Español

Polydentia no garantiza la compatibilidad con otras sustancias distintas de las enumeradas en la tabla.